

JustStop

Topical Soluble Hemostatic Dressing

Size	Units Per Box
0.76" x 0.76" Size: 1.9cm x 1.9cm	20

Manufactured by

HUIZHOU FORYOU MEDICAL DEVICES CO., LTD
North Shangxia Rd., Dongjiang Hi-tech Industry Park,
516005, Huizhou, P.R.China
TEL +86-752-5302185 FAX +86-752-5302020

Manufactured for

Point Blank Medical Pty. Ltd.
F12, 1-15 Barr Street Balmain NSW 2041
Tel: 1300 001 726

JustStop®

Topical Soluble Hemostatic Dressing

Description

JustStop® Topical Soluble Hemostatic Dressing is a white or light yellow sterile fabric prepared by the chemical treatment of regenerated cellulose. It is an effective and innovative soluble hemostat widely use in first-aid and surgery hemostasia. When the fabric comes in contact with blood or exudates, it can quickly transform into a clear viscous gel, which fills wound voids, seals capillary ends, activates the clotting system and thereby help to stop bleeding.

Indications

Prescription:

JustStop® is indicated for the management of topical wounds and to temporary control of external surface bleeding.

Over the Counter (OTC)

JustStop® is indicated for temporary external topical bleeding of minor cuts, minor laceration.

Contraindications

JustStop® Soluble Hemostatic Dressing should not be used for large area of wound hemostasis and hemorrhage of large arteries.

Directions for use

- 1.Clean the wound.
- 2.Apply JustStop® to cover the bleeding site. It can be cut or folded as needed.
- 3.Gently press the wound site until bleeding stops.
- 4.Moisten JustStop® with clean water after the bleeding stops.
- 5.Excess JustStop® gel can be rinsed away with clean water.
- 6.Repeat the steps above with another piece of JustStop® as needed. If bleeding continues, contact a Doctor or Emergency Room.

Warning

- In case of wound swelling or redness for more than 48 hours after use of JustStop® contact a doctor.
- For external topical and temporary use only.

Caution

JustStop® is supplied sterile. Damaged packaging products should not be used.

JustStop® should not be used on dirty, contaminated or infected wounds. Consult your doctor if you notice any itching, burning, rash or signs of infection such as fever, pain, redness or swelling.

Packages

JustStop® is available sterile-packed individually in different sizes.

Storage

Keep JustStop® in a cool dry place for maximum shelf life. The preferred storage temperature should be between 5°C to 40°C (41 °F to 104 °F).










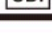


Sterile

JustStop® is sterilized in the original packaging by gamma radiation.

All gauze are sterilized and sold directly to users after sterilization by radiation using conditions validated following ISO 11137-2

Shelf life

Three years.

Symbols used on labelling	
	Do not re-use
	Use-by date
	Temperature limit
	Batch code
	Catalogue number
	Date of manufacture
	Sterilized using irradiation
	Caution
	Do not use if package is damaged
	Do not re-sterilize
	Manufacturer
	Unique device identifier